


Kliniderm[®] **super fiber**

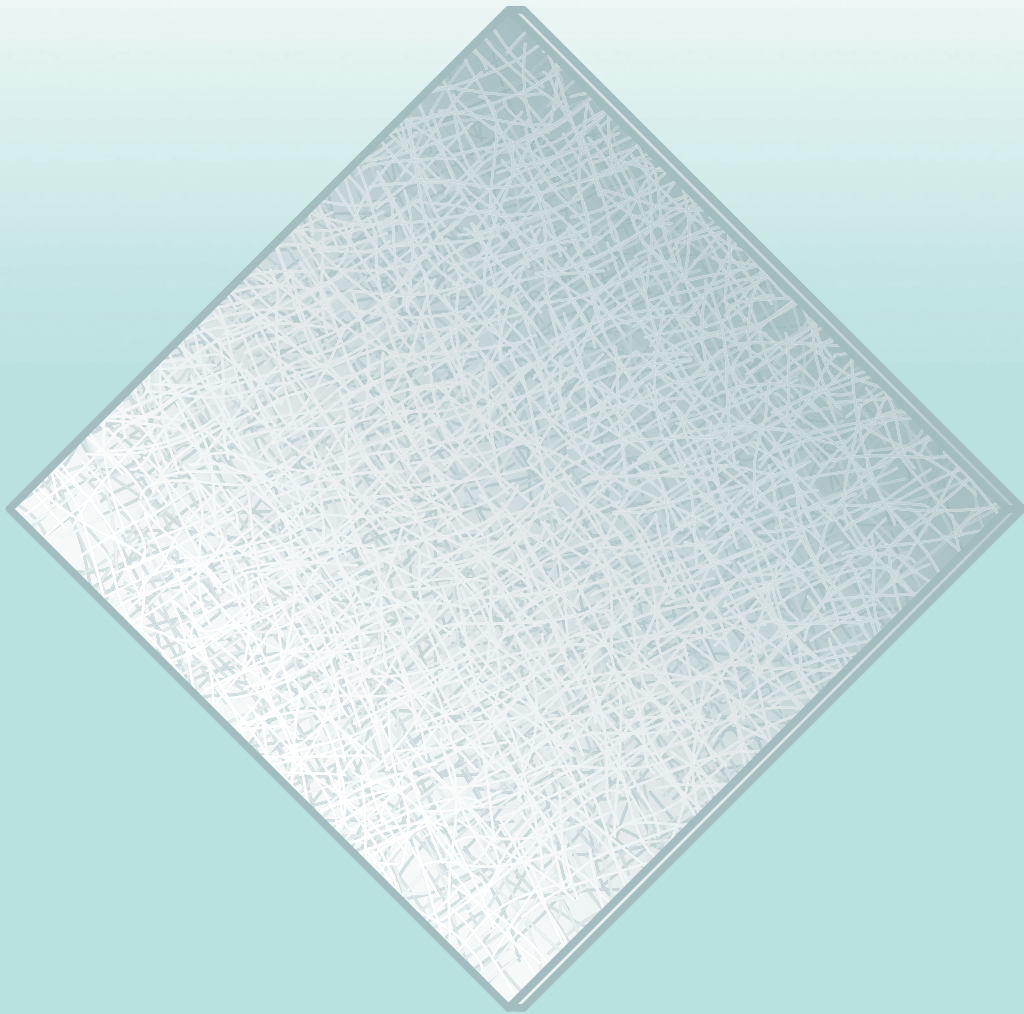

Strong • Absorbent • Conformable

Kliniderm® super fiber

Kliniderm® covers the advanced products of Klinion - a well - known wound care brand. For over 100 years, Klinion offers patients global good quality products for competitive prices. With a wide range, Kliniderm® products ensure optimal user comfort. We are excited to announce our Kliniderm® Foam Super Fiber range.

Kliniderm® Super Fiber is a single-use sterile, non-woven wound dressing composed of 100% sodium carboxymethylcellulose fibers. This adaptable and highly absorbent dressing absorbs wound fluid and turns into a cohesive transparent gel. This maintains a moist environment to support the wound healing process and minimizes patient pain and discomfort during dressing changes or when the dressing is in place.

The dressing absorbs and retains wound fluid, isolating harmful components in wound fluid such as bacteria and proteinases. This reduces the risk of wound infection and maceration. The dressing can be used as a primary dressing for moderately to heavily exuding red wounds.

Indications

Kliniderm® Super Fiber is suitable for chronic and acute wounds with moderate to high wound fluids where the epidermis or dermis has been breached. The dressing can be used throughout the healing process on: Leg ulcers, pressure ulcers (stage II-IV), diabetic ulcers, surgical wounds, donor sites, secondary healing wounds, second degree burns and trauma wounds.

Strong, absorbent and conformable



Super strength

The special design provides high tensile strength, allowing the dressing to be removed in one piece



Conformable

The cohesive gel provides optimal contact with the wound bed



Absorbs and manages exudate

To protect surrounding skin and support moist wound healing



Up to 7 days wear time

To minimise dressing changes



Skin friendly

The cohesive transparent gel minimizes pain and discomfort to the patient



Can be cut to shape

To suit clinical requirements



Medeco B.V.
Brandpuntlaan Zuid 14
2665 NZ Bleiswijk
The Netherlands



www.medeco.org

KWC_01_20250206