# klinidur® long stretch bandage



INSTRUCTIONS FOR USE

**KLINION®**

Wernli AG  
Eggasse 4  
CH-4852 Rothrist  
Switzerland

Distributed by  
Medeco B.V.  
Alexander Flemingstraat 2  
NL-3261 MA Oud-Beijerland  
The Netherlands

IFU\_KWC055\_03  
05-2019

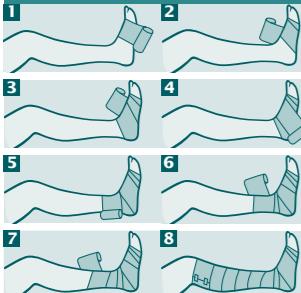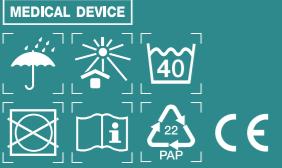## ENGLISH

### Product description

Klinidur® is an elasticated long-stretch heavy duty bandage. Klinidur® provides constant compression and sturdy support to fractures and ruptured or sprained tendons or ligaments. In addition, the bandages are suitable for compression in patients with vascular diseases. The bandages must be removed at night because they exert continual venous pressure.

Klinidur® long stretch bandage consists of 93% cotton, and 7% polyurethane. The material is breathable, durable and resistant to ointments and perspiration.

Medical device.

### Contra-indications

- Do not apply directly onto open wounds.
- Do not use directly on damaged skin.

### Instructions

Bandaging technique:  
Use a circular bandaging technique. The bandaging method described below involves winding the bandage around the leg.

- Select the correct bandage width, based on the diameter of the leg which is to be bandaged.
- When applying the bandage, the foot should generally be kept at a 90° angle with respect to the leg.
- Make sure that the bandage is not under too much tension when applied to avoid shutting off the circulation.
- When bandaging, the roll should be kept as close to the leg as possible.

- Start with the Klinidur® long stretch bandage in the middle of the sole of the foot, just covering the big toe joint (fig. 1)
- Wind it once around the middle of the heel, and then around the lower part of the heel (fig. 2 to 4)
- Wind the bandage upwards around the calf until just below the knee, and fix it at this point with a Kliniplast® surgical tape fixing plaster (fig. 5 to 8)

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