klinidur®

short stretch bandage

INSTRUCTIONS FOR USE

KLINION®

Medeco B.V.

Alexander Flemingstraat 2


NL-3261 MA

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The Netherlands

www.medeco.org

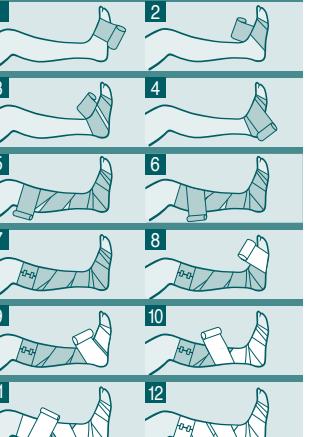IFU_KWC054_02
03-10-2019 / 1_030103

ENGLISH

Product description

Klinidur® short stretch bandage is a lengthwise elasticated, beige colored, breathable short stretch compression bandage. It is made of 100% cotton.

Not made with natural rubber latex. Medical device.

Indications

- Klinidur® short stretch bandage is suitable for giving ambulant compression therapy to patients with vascular diseases, to reduce edema.
- Ambulant compression therapy provides high working pressure and low resting pressure on the lower leg, consequently the bandage can remain in place overnight. For this treatment, frequent leg movement is important.
- For these Behandlung ist eine regelmäßige Beugung der Beine wichtig.

Contra-indications

- Do not apply directly on open wounds or breached skin.

Directions for use

The compression therapy showed in the pictures requires two short stretch bandages that are applied to the lower leg in opposite directions. You can use one bandage, depending on your local protocol.

- Use a circular bandaging technique.
- Select the correct bandage width, based on the diameter of the leg which is to be bandaged.
- Product length is based on fully stretched bandage.
- When applying the bandage, the foot should generally be kept at a 90° angle with respect to the leg.
- Always bandage around the heel; there should not be any openings over the heel.
- Ensure the leg is sufficiently padded to prevent blisters, weals and pressure points.
- When bandaging keep the bandage stretched and the roll short.
- Secure the bandage with an adhesive plaster.

Application:

The bandaging method described below involves winding the bandage around the leg.

1. Start with the first Klinidur® short stretch bandage at the big toe and bring this over the top of the foot towards the little toe.
2. Then bring the bandage once over the middle of the foot thereby securing the bandage.
3. Now bandage towards the middle of the heel whereby the bandage is turned for two-thirds around the side of the heel and one-third underneath it.
4. Then bandage the still exposed part of the foot with one single turn.
5. Next, apply the bandage in one turn horizontally above the ankle and then carry on bandaging around the leg to just under the knee.
6. Just under the knee-cap make one singular turn with the bandage and then bandage down the leg covering any exposed points.
7. Finally, bandage upwards again. Secure the bandage with two pieces of adhesive plaster.
8. Steps 8 to 12 (only if you use the 2 bandage technique) Repeat steps 1 to 7 with the second Klinidur® short stretch bandage, but in the opposite direction. Start bandaging at the little toe and work in the direction of the big toe.

Precautions

- Do not apply with too much tension, wrapping too tightly may impair circulation.
- Please wash with similar colors, airy flat.
- Bandage clips are not intended to be applied on patients.

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