
klinidur® compression bandage forte compression bandage

INSTRUCTIONS FOR USE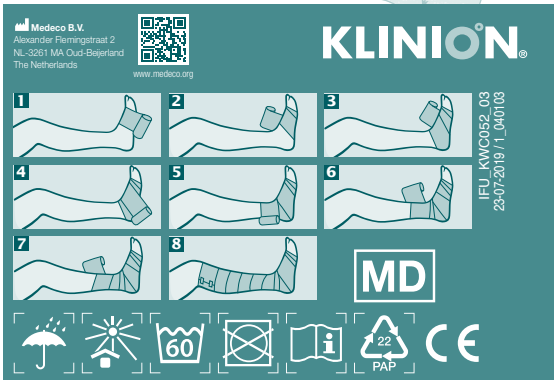

ENGLISH

Product Description

Klinidur® is an elasticated long-stretch bandage which is available in a light (Standard) and a heavy duty (Forte) version. Klinidur® provides constant compression and sturdy support to fractures and ruptured or sprained tendons or ligaments. Klinidur® Standard consists of 82% cotton, 13% polyamide and 5% elastane. The material is breathable, durable and resistant to perspiration. Klinidur® Forte consists of 85% cotton, 7% polyamide and 8% elastane. The material is breathable, durable and resistant to perspiration.

MD Medical device.

Indications

- The bandages are suitable for compression in patients with vascular diseases.
- The bandages must be removed at night because they exert continuous venous pressure.

Contra-indications

- Do not apply directly on open wounds or breached skin.

Directions for use

- Use a circular bandaging technique.
- Select the correct bandage width, based on the diameter of the leg which is to be bandaged.

- Product length is based on fully stretched bandage.
- When applying the bandage, the foot should generally be kept at a 90° angle with respect to the leg.
- Make sure that the bandage is not under too much tension when applied to avoid shunting of the circulation.
- When bandaging, the roll should be kept as close to the leg as possible.
- The bandaging method described below involves winding the bandage around the leg.

Application:

1. Start with the Klinidur® long-stretch bandage in the middle of the sole of the foot, just covering the big toe joint (fig. 1).
2. Wind it once around the middle of the heel and then around the lower part of the heel, followed by the upper part of the heel (fig. 2 to 4).
3. Wind the bandage upwards around the calf until just below the knee and then fix it at this point with a Kliniplast Textile fixing plaster (fig. 5 to 8).

Precautions

- Please wash with similar colors, air dry flat.
- Bandage clips are not intended to be applied on patients.

In case a serious incident has occurred in relation to the device: Inform Medeco BV as listed on the packaging, your local distributor and your local healthcare authority.

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