klinidur textelast®

INSTRUCTIONS FOR USE

KLINION®

IFU_KWC050_03
24-09-2019 / 1_030103

ENGLISH

Product description

Klinidur Textelast is a lightweight elasticated, heavy quality, beige colored, short stretch compression bandage. It is made of 100% cotton; breathable, resistant to perspiration and permeable to x-rays.

Not made with natural rubber latex. Medical device.

Indications

- Klinidur textelast is suitable for giving ambulant compression therapy to patients with vascular diseases, to reduce edema.
- Ambulant compression therapy provides high working pressure and low resting pressure on the lower leg, consequently the bandage can remain in place overnight.
- For this treatment, frequent leg movement is important.

Contra-indications

- Do not apply directly on open wounds or breached skin.

Directions for use

- Use a circular bandaging technique.
- Select the correct bandage width, based on the diameter of the leg which is to be bandaged.
- Product length is based on fully stretched bandage.
- When applying the bandage, the foot should generally be kept at a 90° angle with respect to the leg.
- Always bandage around the heel; there should not be any openings over the heel.
- Ensure the leg is sufficiently padded to prevent blisters, weals and pressure points.
- When bandaging keep the bandage stretched and the roll short.

- Secure the bandage with an adhesive plaster.

Application:
The bandaging method described below involves winding the bandage around the leg.

- Start with the first Klinidur Textelast short stretch bandage at the big toe and bring this over the top of the foot towards the little toe.
- Then bring the bandage once over the middle of the foot thereby securing the bandage.
- Now bandage towards the middle of the heel whereby the bandage is turned for two-thirds around the side of the heel and one-third underneath it.
- Then bandage the still exposed part of the foot with one single turn.
- Next, apply the bandage in one turn horizontally above the ankle and then carry on bandaging around the leg to just under the knee.
- Just under the knee-cap make one singular turn with the bandage and then bandage down the leg covering any exposed parts.
- Finally, bandage upwards again. Secure the bandage with two pieces of adhesive plaster.
- If you use the two-bandage technique: repeat step 1 to 7 with the second Klinidur Textelast short stretch bandage, but in the opposite direction. Start bandaging at the little toe and work in the direction of the big toe.

Precautions

- Do not apply with too much tension, wrapping too tightly may impair circulation.
- Please wash with similar colors, air dry flat.
- Bandage clips are not intended to be applied on patients. In case a serious incident has occurred in relation to the device: Inform Medeco BV as listed on the packaging, your local distributor and your local healthcare authority.

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