# klinigrip® cohesive support bandage


INSTRUCTIONS FOR USE

## KLINION®

Medeco B.V.  
Alexander Flemingstraat 2  
NL-3261 MA Oud-Beijerland  
The Netherlands

www.medeco.org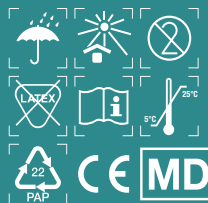

## ENGLISH

**Product description**  
Klinigrip® cohesive support bandage consists of 87% cotton, 12% synthetic rubber and 1% elastane. The bandage is lengthwise elasticated, breathable, beige colored, comfortable and includes an end-finder. Klinigrip® cohesive support bandage does not stick to skin, hair or clothing.†

Not made with natural rubber latex. Medical device.

### Indications

- Klinigrip® cohesive support bandage is suitable for the treatment of musculoskeletal injuries and for post-fracture treatment.
- Klinigrip® cohesive support bandage can be used for support, relief, fixation and compression.

### Contra-indications

- Do not apply directly on open wounds or breached skin.

### Directions for use

- Use a circular bandaging technique.
- Select the correct bandage width, based on the diameter of the leg which is to be bandaged.
- Product length is based on fully stretched bandage.
- Klinigrip® cohesive support bandage can be torn to enable easy and rapid application, with economical use of the material. The bandage is intended for single patient use.
- The elasticity and cohesive strength of the bandage makes sure it stays in place without using securing plasters.

### Precautions

- Do not apply with too much tension, wrapping too tightly may impair circulation.
- Do not re-use. Single patient use only.
- Storage temperature should not exceed 25°C.

In case a serious incident has occurred in relation to the device: Inform Medeco BV as listed on the packaging, your local distributor and your local healthcare authority.

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