# L-Mesitran® NET Hydro-active, honey-hydrogel coated, open weave mesh wound dressing

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### L-Mesitran Net

#### Product description

L-Mesitran Net is a sterile, open-weave mesh dressing coated with a hydro-active honey-hydrogel for use on acute and chronic wounds. L-Mesitran Net contains 20% medical grade honey, an acrylic polymer gel and water on a polyester mesh structure that allows passage of exudate through the dressing. L-Mesitran Net is to be used in conjunction with an appropriate secondary dressing, and can be used in conjunction with L-Mesitran Ointment and Soft. L-Mesitran Net is a multipurpose protective wound contact layer, where the honey-hydrogel coating provides some absorption and retention of exudate to help maintain a moist environment conducive to healing.

#### Indications

L-Mesitran Net is indicated for low to heavily exuding chronic wounds such as pressure ulcers; superficial and partial thickness burns; venous, arterial and diabetic ulcers; fungating wounds. It is also indicated for acute wounds such as donor sites, surgical wounds, cuts and abrasions.

#### Contraindications

- Do not use on individuals with a known sensitivity to the dressing or its components. L-Mesitran Net should not be used.
- Without a secondary dressing to prevent the dressing from drying out on the wound and to retain the dressing on the wound.
- On its own on heavily exuding wounds.
- On full thickness burns, deep, narrow cavities or sinuses.

#### Warnings and precautions

The dressing may be used on contaminated and colonized wounds. However where wounds are clinically infected, L-Mesitran Net should only be used providing appropriate medical supervision is sought. Any further treatment for infection should be initiated as per local protocols and under medical guidance. Appropriate secondary dressings should be used to prevent the dressing from drying out whilst in place. Do not use if the packaging is damaged or compromised in any way.

#### Instructions for use

- Inspect and if necessary, cleanse the wound as per local protocols.
- Remove the sterile dressing from the pouch.
- Peel the dressing off the clear plastic backing sheet and apply the freshly-exposed side to the wound.
- Cover and secure with the appropriate secondary dressing.
- Inspect the wound and the surrounding tissue regularly, as per local protocols.
- In the event of an intolerance to the dressing or one of its components, remove the dressing using appropriate techniques and clean the area carefully.

#### Frequency of change

The dressing can remain in place for up to five days, dependent upon the amount of wound exudate and the needs of the wound. Where appropriate, L-Mesitran Net may remain in place while the secondary dressing is changed. Complete and gentle removal of the dressing may be aided by irrigation should the dressing

become too dry. Each dressing is for single use. Do not use after expiry date.

Store between 5°C and 25°C. Avoid direct sunlight and moisture.

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© Theo Manufacturing BV, November 2014
**Theo Manufacturing B.V.**