L-Mesitran® Soft - Hydroactive antibacterial wound gel

NEDERLANDS

L-Mesitran® Soft hydroactieve wondgel

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L-Mesitran® Soft hydroactive wound gel

Product description

L-Mesitran Soft is a gel that contains: medical grade honey, medical grade hyaloergenic lanolin (Medilan®), propylene glycol, PEG 4000, and vitamins C & E. When the gel is in contact with the wound, fluid is drawn from the surrounding tissues through osmosis. By doing so, a moist wound healing environment is created. This environment stimulates the wound healing process, as wounds bed preparation by facilitating autolysis of necrotic and other devitalised material and promotes better epithelial cell migration.

- Due to the strong osmotic activity L-Mesitran Soft has antimicrobial properties.
- The gel will dilute gradually and can be removed easily as per local wound cleansing protocols.
- L-Mesitran Soft is a primary wound dressing that can be covered with most commonly used secondary dressings.
- The properties of the gel also help prevent the secondary dressing adhering to the wound bed.
- Due to the use of natural ingredients, each batch of gel may vary in colour (white - yellow - brown).
- The gel may contain sugar crystals, however this will not affect the effectiveness and/or application of the gel.

Indications

Chronic wounds, pressure ulcers (Stage I-IV), venous stasis ulcers, fungating wounds (to help deodorise and/or debride). Colonised acute wounds, surgical wounds, post-operative wounds, traumatic wounds (superficial wounds, cuts). Small burn and laser wounds (superficial and partial thickness burns).

ANWENDING

Contraindications
Do not use the gel on patients who are known to be hypersensitive to it or any of its ingredients. No other contraindications are known to date.

Preventive measures and comments

- When opening and applying the gel from a new tube, ensure that the new tube is not damaged or previously opened.
- Consult a doctor or pharmacist if further measures are deemed necessary for proper treatment of the wound.
- Inspect and clean the wound according to local protocols before use.
- Due to autolytic debridement the wound may appear to increase in size after the first dressing changes. This is normal and occurs as the devitalised tissue is removed, preparing the wound bed for healing.
- Bacterial colonisation of chronic wounds is always present and is not a contraindication for using L-Mesitran Soft.
- Seek medical supervision if signs of infection occur (fever, pain, redness and pus) or if the condition of the wound deteriorates.

- L-Mesitran Soft is for use by a single patient only. The tube may be re-used with the same patient.
- Always re-seal the tube after use with the screw-top lid.
- After the packaging has been opened, provided the product is used in accordance with the instructions described in this leaflet, the storage life of the product is equal to the expiry date printed on the packaging.

Date of opening:/...../20

Instructions for use

- Remove lid, clean the mouth of the tube and the top of the lid with an alcohol swab.
- On 1st use of a tube, it is necessary to pierce the opening of the tube by pushing the top of the lid into the foil seal covering the nozzle. Avoid squeezing the tube at this time.

Application to the wound:

- Keeping the nozzle clear of the wound surface, gently squeeze the tube, applying a sufficient layer of the gel to the whole wound area (including the peri wound area).
- Use a (sterile) spatula if necessary to distribute the gel over the tube wound surface.
- Cover the wound with an appropriate secondary dressing.
- Re-apply the gel to the wound as soon as the gel has disappeared. This depends on the rate and volume of fluid produced by the wound.

Storage precautions

Store at room temperature 5°-25°C / 41°-77°F. L-Mesitran Soft is a natural product and may become liquid under varying temperatures. If this is expected to be an issue, then the tube may be placed in a fridge prior to use.

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Contraindicaciones
Do not use the gel on patients who are known to be hypersensitive to it or any of its ingredients. No other contraindications are known to date.

Remove lid, clean the mouth of the tube and the top of the lid with an alcohol swab.

- On 1st use of a tube, it is necessary to pierce the opening of the tube by pushing the top of the lid into the foil seal covering the nozzle. Avoid squeezing the tube at this time.

Application to the wound:
Keeping the nozzle clear of the wound surface, gently squeeze the tube, applying a sufficient layer of the gel to the whole wound area (including the peri wound area).

- Use a (sterile) spatula if necessary to distribute the gel over the tube wound surface.
- Cover the wound with an appropriate secondary dressing.
- Re-apply the gel to the wound as soon as the gel has disappeared. This depends on the rate and volume of fluid produced by the wound.

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